

# Safety Retractable Syringes

## Safety Syringe Comprehensive Solution



### Product Specification

Description	Syringe Volume	Needle Gauge	Luer connector
Vaccine Safety Syringe	0.5ML/1ML 3ML/5ML	23G, 25G (16mm-25mm)	Integrated Luer lock
Insulin Safety Syringe	0.5ML/1ML	27G/29G/30G/31G (8mm,12.7mm)	Integrated
Hypodermic Safety Syringe	1ML 3ML/5ML/10ML	18G/19G/21G/22G/23G/25G (16mm-38mm)	Integrated Luer lock




#### 1→4 How To Use



- Draw up medication.
- Inject medication, and the safety function will lock in when the plunger is fully depressed at the bottom to the barrel.
- Remove from patient's body and retract plunger until the needle is pulled back into the syringe.
- Pull the plunger all the way to the end of barrel, and snap off the plunger to discard the used syringe in a sharp disposal container.

# Safety Clip Syringes

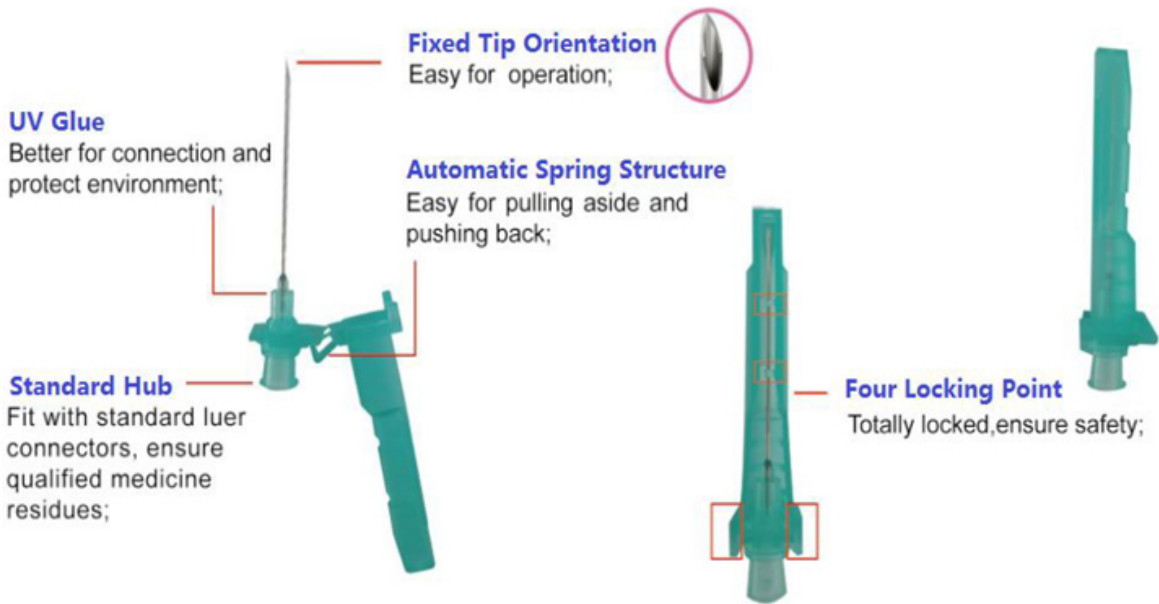

## Product Specification

Description	Needle Gauge	Needle Length	
Safety Hypodermic Needle	16G—30G	5/16"—1-1/2"(8mm-40mm)	
Description	Syringe Volume	Needle Gauge	Luer connector
Safety Clip Syringes	1ML	18G—30G(0.3mm-1.2mm)	Luer slip
	2ML/2.5ML/3ML	18G—30G(0.3mm-1.2mm)	Luer slip/luer lock
	5ML	18G—30G(0.3mm-1.2mm)	Luer slip/luer lock
	10ML	18G—30G(0.3mm-1.2mm)	Eccentric/Centric luer slip/lock
	20ML	18G—30G(0.3mm-1.2mm)	Eccentric/Centric luer slip/lock



## Instruction for Use



1 Peel off the blister; 2 Take off the safety needle, and rotate into the syringe; 3 Take off the needle protector; 4 Drawing the medicine liquid to exact "ML" required; 5 Injecting into the human body; 6 Press the safety sheath to lock the needle.

