**EE**  
3M™ Aura™ Tolmurespirator

**HR**  
3M™ Aura™ Respirator za čestice

**LV**  
3M™ Aura™ Dajlinu respirators

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3M United Kingdom PLC, Cain Road, Bracknell, Berkshire, RG12 8HT, UK

3M Wroclaw Sp. z o.o., Kowalska 143, 51-424 Wroclaw, Poland

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## WARNINGS AND LIMITATIONS

Allways be sure that the complete product is:

- Suitable for the application;

- Fitted correctly;

- Worn during all periods of exposure;

- Repaired when necessary;

- Proper selection, training, use and appropriate maintenance are essential in order for the product to help protect the wearer from certain airborne contaminants. Failure to follow all instructions on the use of these respiratory protection products and/or failure to properly wear the product produced during all periods of exposure may result in serious health effects to the wearer;

- Due to life threatening risk of permanent disability;

- For suitable and proper use follow local regulations and refer to all information supplied. For more information contact a safety professional or 3M on 0870 60 800 60 (UK) or 100 320 500 (Ireland);

- Before use, the wearer must be trained in use of the complete product in accordance with applicable Health and Safety standards/guidance;

- These products do not contain components made from natural rubber latex;

- Do not use in atmospheres containing less than 19.5% oxygen. (3M definition). Individual countries may apply their own limits on oxygen deficiency. Seek advice if in doubt;

- Do not use for respiratory protection against atmospheric contaminants/concentrations which are unknown or immediately dangerous to life and health (IDLH);

△ Do not use with beards or other facial hair that may inhibit contact between the face and the product thus preventing a good seal.

- Leave the contaminated area immediately if:

a) Breathing becomes difficult;

b) Dizziness or other distress occurs;

c) Discard and replace the respirator if it becomes damaged, breathing resistance becomes excessive or at the end of the shift;

d) Do not alter, modify, clean or repair this respirator;

e) Dispose in accordance with local regulations;

f) In case of intended use in explosive atmospheres, contact 3M;

g) Before initial use, always check that the product is within the stated shelf life (use by date);

## FITTING INSTRUCTIONS

See Figure 1.

Before fitting device, ensure hands are clean.

All respirator components should be tested for damage prior to each use.

1. With reverse side up and using tab, separate top and bottom panels to form a cup shape.

Bend slightly at the centre of the noseclip.

2. Ensure both panels are fully unfolded.

3. Cup respirator in one hand with open side towards face. Take both straps in other hand. Hold respirator under chin, with nosepiece up, and pull straps over head.

4. Locate the upper strap across the crown of the head and the lower strap below the ears. Straps must not be twisted.

Adjust top and bottom panels for a comfortable fit, ensuring panels and tab are not folded in.

5. Using both hands, mould noseclip to the shape of the nose to ensure a close fit and good seal. Pinching the noseclip using only one hand may result in less effective respirator performance.

6. The seal of the respirator on the face should be fit-checked before entering the workplace.

## FIT CHECK

1. Cover the front of the respirator with both hands being careful not to disturb the fit of the respirator;

2a) UNVALVED respirator - EXHALE sharply;

2b) VALVED respirator - INHALE sharply;

3. If air leaks around the nose, re-adjust the noseclip to eliminate leakage.

Repeat until fit checked.

If air leak is at the respirator edges, work the straps back along the sides of the head to eliminate leakage. Repeat the above fit check.

If you CANNOT achieve a proper fit DO NOT enter the hazardous area.

See your supervisor.

Users should be fit tested in accordance with national requirements.

For information regarding fit procedures, please contact 3M.

## STORAGE AND TRANSPORTATION

Store in accordance with manufacturer's instructions, see packaging.

**NOTE** Average conditions may exceed 25°C / 80% RH for short periods.

The maximum shelf life of 3 years applies to this item for no more than 3 months in the shelf life of the product. When storing or transporting this product use original packaging provided. Do not store in direct sunlight.

## MARKING

NR = Non-reusable (single use only) D= Meets the clogging requirements

□ = End of Shelf Life Date format: YYYY/MM/DD □ = Temperature

Range □ = Maximum Relative Humidity □ = Dispose in accordance with local regulations. □ = Packaging not suitable for food contact.

## APPROVALS

The products are type approved and audited annually by either: BSI Group, The Netherlands B.V. Say Building, John M. Keynesplein 9, NL-1066 EP Amsterdam, The Netherlands, Body No. 2797 and/or BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowhill, Milton Keynes MK5 8PP, UK, Notified Body No. 0086. These products meet the requirements of European Regulation (EU) 2016/425 and applicable local legislation. The applicable European/local legislation and Notified Body can be determined by reviewing the Certificate(s) and Declaration(s) of Conformity at [www.3M.com/Respiratory/certs](http://www.3M.com/Respiratory/certs).

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